

DAILIES® AquaComfort PLUS® Toric



HOW TO USE THE DAILIES® AquaComfort PLUS® TORIC CONTACT LENS SELECTION GRID

NOTE: There is no need to correct for vertex distance, as the chart will do this for you.

1. Locate the sphere power in the spectacle refraction on the y axis of the grid labeled "Spectacle Sphere Powers."
2. Locate the DAILIES® AquaComfort PLUS® Toric trial contact lens power underneath the spectacle minus cylinder power in the spectacle Rx.
3. For trial contact lenses, select the DAILIES® AquaComfort PLUS® Toric contact lens axis closest to the refractive axis.

Vertex Correction: 12mm

SPECTACLE CYLINDER POWERS (D)

SPECTACLE SPHERE POWERS (D)	-0.75		-1.00		-1.25		-1.50		-1.75		-2.00		-2.25		-2.50		-2.75		-3.00	
	sph	cyl	sph	cyl	sph	cyl	sph	cyl	sph	cyl	sph	cyl	sph	cyl	sph	cyl	sph	cyl	sph	cyl
-10.50			-9.00	-0.75	-9.00	-0.75	-9.00	-1.25	-9.00	-1.25	-9.00	-1.25	-9.00	-1.75	-9.00	-1.75				
-10.25			-9.00	-0.75	-9.00	-0.75	-9.00	-1.25	-9.00	-1.25	-9.00	-1.25	-9.00	-1.75	-9.00	-1.75				
-10.00	-9.00	sph	-9.00	-0.75	-9.00	-0.75	-9.00	-1.25	-9.00	-1.25	-9.00	-1.25	-9.00	-1.75	-9.00	-1.75				
-9.75	-9.00	sph	-8.50	-0.75	-8.50	-0.75	-8.50	-1.25	-8.50	-1.25	-8.50	-1.25	-8.50	-1.75	-8.50	-1.75				
-9.50	-8.50	sph	-8.50	-0.75	-8.50	-0.75	-8.50	-1.25	-8.50	-1.25	-8.50	-1.25	-8.50	-1.75	-8.50	-1.75				
-9.25	-8.50	sph	-8.50	-0.75	-8.50	-0.75	-8.50	-1.25	-8.50	-1.25	-8.50	-1.25	-8.50	-1.75	-8.50	-1.75				
-9.00	-8.00	sph	-8.00	-0.75	-8.00	-0.75	-8.00	-1.25	-8.00	-1.25	-8.00	-1.75	-8.00	-1.75	-8.00	-1.75	-8.00	-2.25	-8.00	-2.25
-8.75	-8.00	sph	-8.00	-0.75	-8.00	-0.75	-8.00	-1.25	-8.00	-1.25	-8.00	-1.75	-8.00	-1.75	-8.00	-1.75	-8.00	-2.25	-8.00	-2.25
-8.50	-8.00	sph	-7.50	-0.75	-7.50	-0.75	-7.50	-1.25	-7.50	-1.25	-7.50	-1.75	-7.50	-1.75	-7.50	-1.75	-7.50	-2.25	-7.50	-2.25
-8.25	-7.75	sph	-7.50	-0.75	-7.50	-0.75	-7.50	-1.25	-7.50	-1.25	-7.50	-1.75	-7.50	-1.75	-7.50	-1.75	-7.50	-2.25	-7.50	-2.25
-8.00	-7.75	sph	-7.50	-0.75	-7.50	-0.75	-7.50	-1.25	-7.50	-1.25	-7.50	-1.75	-7.50	-1.75	-7.50	-1.75	-7.50	-2.25	-7.50	-2.25
-7.75	-7.50	sph	-7.00	-0.75	-7.00	-0.75	-7.00	-1.25	-7.00	-1.25	-7.00	-1.75	-7.00	-1.75	-7.00	-1.75	-7.00	-2.25	-7.00	-2.25
-7.50	-7.25	sph	-7.00	-0.75	-7.00	-0.75	-7.00	-1.25	-7.00	-1.25	-7.00	-1.75	-7.00	-1.75	-7.00	-1.75	-7.00	-2.25	-7.00	-2.25
-7.25	-7.00	sph	-6.50	-0.75	-6.50	-0.75	-6.50	-1.25	-6.50	-1.25	-6.50	-1.75	-6.50	-1.75	-6.50	-1.75	-6.50	-2.25	-6.50	-2.25
-7.00	-6.75	sph	-6.50	-0.75	-6.50	-0.75	-6.50	-1.25	-6.50	-1.25	-6.50	-1.75	-6.50	-1.75	-6.50	-1.75	-6.50	-2.25	-6.50	-2.25
-6.75	-6.50	sph	-6.00	-0.75	-6.00	-0.75	-6.00	-1.25	-6.00	-1.25	-6.00	-1.75	-6.00	-1.75	-6.00	-1.75	-6.00	-2.25	-6.00	-2.25
-6.50	-6.00	-0.75	-6.00	-0.75	-6.00	-0.75	-6.00	-1.25	-6.00	-1.25	-6.00	-1.75	-6.00	-1.75	-6.00	-1.75	-6.00	-2.25	-6.00	-2.25
-6.25	-5.75	-0.75	-5.75	-0.75	-5.75	-0.75	-5.75	-1.25	-5.75	-1.25	-5.75	-1.75	-5.75	-1.75	-5.75	-1.75	-5.75	-2.25	-5.75	-2.25
-6.00	-5.50	-0.75	-5.50	-0.75	-5.50	-0.75	-5.50	-1.25	-5.50	-1.25	-5.50	-1.75	-5.50	-1.75	-5.50	-1.75	-5.50	-2.25	-5.50	-2.25
-5.75	-5.50	-0.75	-5.50	-0.75	-5.50	-0.75	-5.50	-1.25	-5.50	-1.25	-5.50	-1.75	-5.50	-1.75	-5.50	-1.75	-5.50	-2.25	-5.50	-2.25
-5.50	-5.25	-0.75	-5.25	-0.75	-5.25	-0.75	-5.25	-1.25	-5.25	-1.25	-5.25	-1.75	-5.25	-1.75	-5.25	-1.75	-5.25	-2.25	-5.00	-2.25
-5.25	-5.00	-0.75	-5.00	-0.75	-5.00	-0.75	-5.00	-1.25	-5.00	-1.25	-5.00	-1.75	-5.00	-1.75	-5.00	-1.75	-5.00	-2.25	-5.00	-2.25
-5.00	-4.75	-0.75	-4.75	-0.75	-4.75	-0.75	-4.75	-1.25	-4.75	-1.25	-4.75	-1.75	-4.75	-1.75	-5.00	-1.75	-4.75	-2.25	-4.75	-2.25
-4.75	-4.50	-0.75	-4.50	-0.75	-4.50	-0.75	-4.50	-1.25	-4.50	-1.25	-4.50	-1.75	-4.50	-1.75	-4.75	-1.75	-4.50	-2.25	-4.50	-2.25
-4.50	-4.25	-0.75	-4.25	-0.75	-4.25	-0.75	-4.25	-1.25	-4.25	-1.25	-4.25	-1.75	-4.25	-1.75	-4.50	-1.75	-4.25	-2.25	-4.25	-2.25
-4.25	-4.00	-0.75	-4.00	-0.75	-4.25	-0.75	-4.00	-1.25	-4.00	-1.25	-4.00	-1.75	-4.00	-1.75	-4.25	-1.75	-4.00	-2.25	-4.25	-2.25
-4.00	-4.00	-0.75	-4.00	-0.75	-4.00	-0.75	-4.00	-1.25	-4.00	-1.25	-4.00	-1.75	-4.00	-1.75	-4.25	-1.75	-4.00	-2.25	-4.00	-2.25
-3.75	-3.75	-0.75	-3.75	-0.75	-3.75	-0.75	-3.75	-1.25	-3.75	-1.25	-3.75	-1.75	-3.75	-1.75	-3.75	-1.75	-3.75	-2.25	-4.00	-2.25
-3.50	-3.50	-0.75	-3.50	-0.75	-3.50	-0.75	-3.50	-1.25	-3.50	-1.25	-3.50	-1.75	-3.50	-1.75	-3.50	-1.75	-3.50	-2.25	-3.75	-2.25
-3.25	-3.25	-0.75	-3.25	-0.75	-3.25	-0.75	-3.25	-1.25	-3.25	-1.25	-3.25	-1.75	-3.25	-1.75	-3.25	-1.75	-3.25	-2.25	-3.50	-2.25
-3.00	-3.00	-0.75	-3.00	-0.75	-3.00	-0.75	-3.00	-1.25	-3.00	-1.25	-3.00	-1.75	-3.00	-1.75	-3.00	-1.75	-3.00	-2.25	-3.25	-2.25
-2.75	-2.75	-0.75	-2.75	-0.75	-2.75	-1.25	-2.75	-1.25	-2.75	-1.25	-2.75	-1.75	-2.75	-1.75	-2.75	-1.75	-2.75	-2.25	-3.00	-2.25
-2.50	-2.50	-0.75	-2.50	-0.75	-2.50	-1.25	-2.50	-1.25	-2.50	-1.25	-2.50	-1.75	-2.50	-1.75	-2.75	-1.75	-2.50	-2.25	-2.75	-2.25
-2.25	-2.25	-0.75	-2.25	-0.75	-2.25	-1.25	-2.25	-1.25	-2.25	-1.75	-2.25	-1.75	-2.25	-1.75	-2.50	-1.75	-2.25	-2.25	-2.50	-2.25
-2.00	-2.00	-0.75	-2.00	-0.75	-2.00	-1.25	-2.00	-1.25	-2.00	-1.75	-2.00	-1.75	-2.00	-1.75	-2.25	-1.75	-2.00	-2.25	-2.25	-2.25
-1.75	-1.75	-0.75	-1.75	-0.75	-1.75	-1.25	-1.75	-1.25	-1.75	-1.75	-2.00	-1.75	-2.00	-1.75	-2.00	-1.75	-1.75	-2.25	-2.00	-2.25
-1.50	-1.50	-0.75	-1.50	-0.75	-1.50	-1.25	-1.50	-1.25	-1.50	-1.75	-1.50	-1.75	-1.75	-1.75	-1.50	-2.25	-1.50	-2.25	-1.75	-2.25
-1.25	-1.25	-0.75	-1.25	-0.75	-1.25	-1.25	-1.25	-1.25	-1.25	-1.75	-1.25	-1.75	-1.50	-1.75	-1.25	-2.25	-1.50	-2.25	-1.50	-2.25
-1.00	-1.00	-0.75	-1.00	-0.75	-1.00	-1.25	-1.00	-1.25	-1.00	-1.75	-1.00	-1.75	-1.25	-1.75	-1.00	-2.25	-1.25	-2.25	-1.25	-2.25
-0.75	-0.75	-0.75	-0.75	-0.75	-0.75	-1.25	-0.75	-1.25	-0.75	-1.75	-0.75	-1.75	-1.00	-1.75	-0.75	-2.25	-1.00	-2.25	-1.00	-2.25
-0.50	-0.50	-0.75	-0.50	-0.75	-0.50	-1.25	-0.50	-1.25	-0.50	-1.75	-0.50	-1.75	-0.75	-1.75	-0.50	-2.25	-0.75	-2.25	-0.75	-2.25
-0.25	-0.25	-0.75	-0.25	-0.75	-0.25	-1.25	-0.25	-1.25	-0.25	-1.75	-0.25	-1.75	-0.50	-1.75	-0.25	-2.25	-0.50	-2.25	-0.50	-2.25
+0.00	+0.00	-0.75	+0.00	-0.75	+0.00	-1.25	+0.00	-1.25	+0.00	-1.75	+0.00	-1.75	-0.25	-1.75	+0.00	-2.25	-0.25	-2.25	-0.25	-2.25
+0.25	+0.25	-0.75	+0.25	-0.75	+0.25	-1.25	+0.25	-1.25	+0.25	-1.75	+0.25	-1.75	0.00	-1.75	+0.25	-2.25	+0.00	-2.25	+0.00	-2.25
+0.50	+0.50	-0.75	+0.50	-0.75	+0.50	-1.25	+0.50	-1.25	+0.50	-1.75	+0.50	-1.75	+0.25	-1.75	+0.50	-2.25	+0.25	-2.25	+0.25	-2.25
+0.75	+0.75	-0.75	+0.75	-0.75	+0.75	-1.25	+0.75	-1.25	+0.75	-1.75	+0.75	-1.75	+0.50	-1.75	+0.75	-2.25	+0.50	-2.25	+0.50	-2.25
+1.00	+1.00	-0.75	+1.00	-0.75	+1.00	-1.25	+1.00	-1.25	+1.00	-1.75	+1.00	-1.75	+0.75	-1.75	+1.00	-2.25	+0.75	-2.25	+0.75	-2.25
+1.25	+1.25	-0.75	+1.25	-0.75	+1.25	-1.25	+1.25	-1.25	+1.25	-1.75	+1.25	-1.75	+1.00	-1.75	+1.25	-2.25	+1.00	-2.25	+1.00	-2.25
+1.50	+1.50	-0.75	+1.50	-0.75	+1.50	-1.25	+1.50	-1.25	+1.50	-1.75	+1.50	-1.75	+1.25	-1.75	+1.50	-2.25	+1.25	-2.25	+1.25	-2.25
+1.75	+1.75	-0.75	+1.75	-0.75	+1.75	-1.25	+1.75	-1.25	+1.75	-1.75	+1.75	-1.75	+1.50	-1.75	+1.75	-2.25	+1.50	-2.25	+1.50	-2.25
+2.00	+2.00	-0.75	+2.00	-0.75	+2.00	-1.25	+2.00	-1.25	+2.00	-1.75	+2.00	-1.75	+1.75	-1.75	+2.00	-2.25	+1.75	-2.25	+1.75	-2.25
+2.25	+2.25	-0.75	+2.25	-0.75	+2.25	-1.25	+2.25	-1.25	+2.25	-1.75	+2.25	-1.75	+1.75	-1.75	+2.25	-2.25	+2.00	-2.25	+2.00	-2.25
+2.50	+2.50	-0.75	+2.50	-0.75	+2.50	-1.25	+2.50	-1.25	+2.50	-1.75	+2.50	-1.75	+2.25	-1.75	+2.25	-2.25	+2.25	-2.25	+2.25	-2.25
+2.75	+2.75	-0.75	+2.75	-0.75	+2.75	-1.25	+2.75	-1.25	+2.75	-1.75	+2.75	-1.75	+2.50	-1.75	+2.75	-2.25	+2.50	-2.25	+2.50	-2.25
+3.00	+3.00	-0.75	+3.00	-0.75	+3.00	-1.25	+3.00	-1.25	+3.00	-1.75	+3.00	-1.75	+2.75	-1.75	+3.00	-2.25	+2.75	-2.25	+2.75	-2.25
+3.25	+3.25	-0.75	+3.25	-0.75	+3.25	-1.25	+3.25	-1.25	+3.25	-1.75	+3.25	-1.75	+3.00	-1.75	+3.25	-2.25	+3.00	-2.25	+3.00	-2.25
+3.50	+3.50	-0.75	+3.50	-0.75	+3.50	-1.25	+3.50	-1.25	+3.50	-1.75	+3.50	-1.75	+3.25	-1.75	+3.50	-2.25	+3.25	-2.25	+3.00	-2.25
+3.75	+3.75	-0.75	+3.75	-0.75	+3.75	-1.25	+3.75	-1.25	+3.75	-1.75	+3.75	-1.75	+3.50	-1.75	+3.75	-2.25	+3.50	-2.25	+3.25	-2.25
+4.00	+4.00	-0.75	+4.00	-0.75	+4.00	-1.25	+4.00	-1.25	+4.00	-1.75	+4.00	-1.75	+3.75	-1.75	+4.00	-2.25	+4.00	-2.25	+3.75	-2.25
	sph	cyl	sph	cyl	sph	cyl	sph	cyl	sph	cyl	sph	cyl	sph	cyl	sph	cyl	sph	cyl	sph	cyl

TRIAL CONTACT LENS POWERS